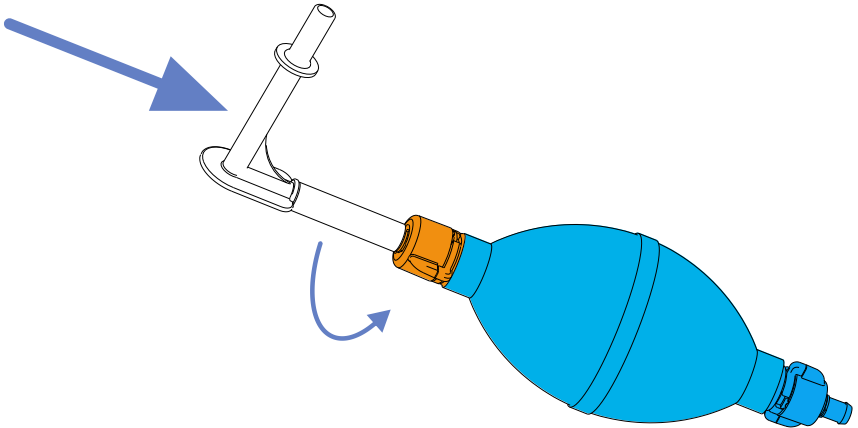
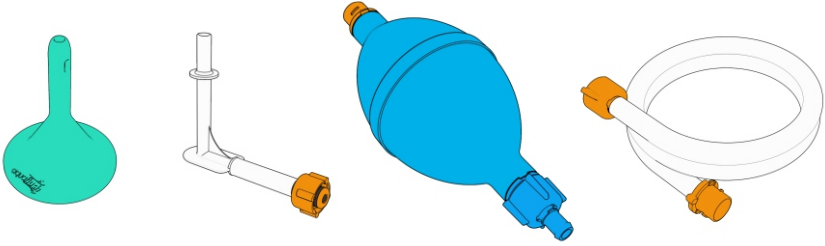
**SOLUTIONS
WITH YOU
IN MIND**

Compact & Compact+

Trans-anal Irrigation Systems: Instructions for Use
Trans-anal Irrigasjonssystem: Brukerveiledning
Transanal irrigations system: Brugsanvisning
Transanal Irrigationssystem: Bruksanvisning
Suolihuuhtelujärjestelmä: Käyttöohje

EN: Thank you for choosing an Aquaflush product. Aquaflush Transanal Irrigation (TAI) systems are designed to be simple, easy to use and an effective way for you to control your bowel management routine with the minimum of fuss. Please read this document carefully, including the warnings and precautions, if you have any questions please contact us or discuss any concerns with your healthcare professional who will also provide you with some training before commencing this treatment.

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1

EN: Wash your hands thoroughly before starting. Connect the elbow to the pump and then push the cone onto the elbow firmly ensuring a tight fit. If you are not able to hover over the toilet the optional extension tube can be used to make it easier whilst seated. Connect the elbow to the extension tube, then connect the extension tube to the pump.

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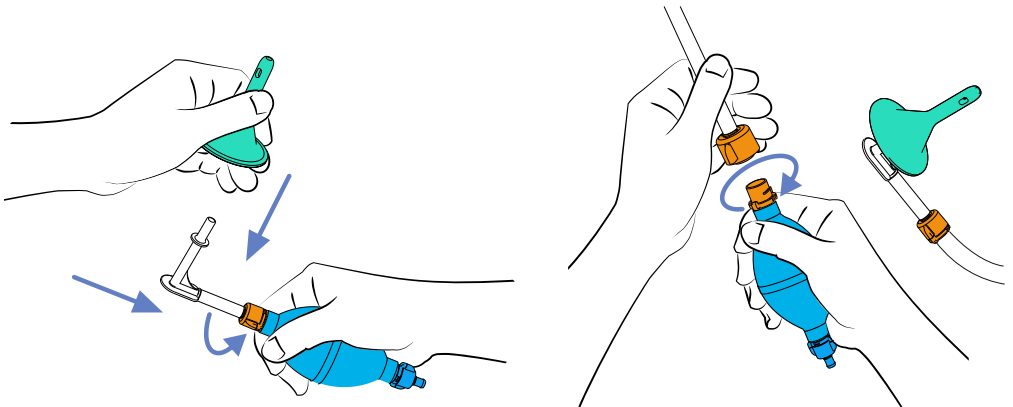
with optional extension tube


EN: Prime the system with clean water between 35-37°C (body temperature) from a bowl or washbasin. If the water is too cold it may cause cramps, if it is too hot there is a danger of trauma to the bowel lining. It is important that you do not pump air into your bowel so always prime the system as described here. Squeeze the pump, then submerge the bottom end in the water, relax your grip which will draw water up into the pump and tubing. Repeat this process until all air is expelled from the system and a steady stream of water comes out of the cone tip.

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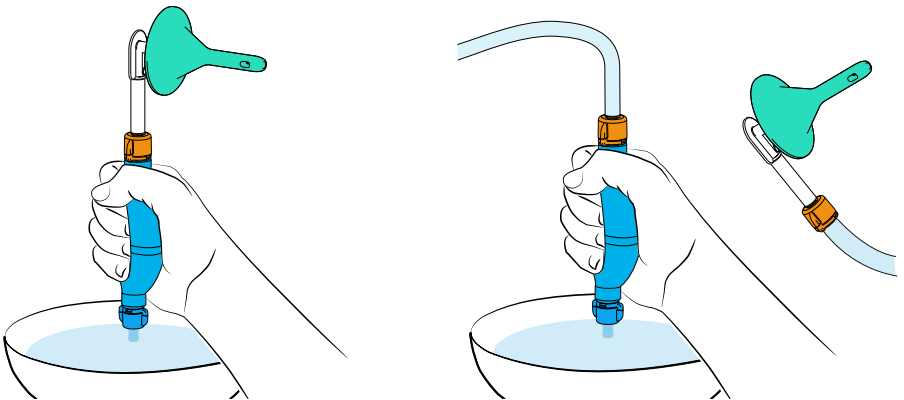EN: Apply the supplied lubricant to the tip and shaft of the cone, you can also apply some to the anus if required. Either while hovering over or seated on the toilet, carefully push the cone into the rectum, some people may need to be taught the correct angle of insertion. Hold the pump at a slight upward angle so any air inside the pump does not get pumped into your bowel. Gently pump the water into your rectum as shown by your prescribing professional. If any discomfort is felt, stop pumping immediately. Once the above process is complete remove the cone from the rectum and remain seated on the toilet. The bowel may take up to 30 minutes to empty, wait about 5 minutes after everything appears to have been expelled.

Optional: some people find that performing a second irrigation after they have had a first motion/emptying [perhaps less than expected] is useful to ensure the descending colon is completely empty. Your healthcare professional will be able to offer guidance on the number of irrigations you require.

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
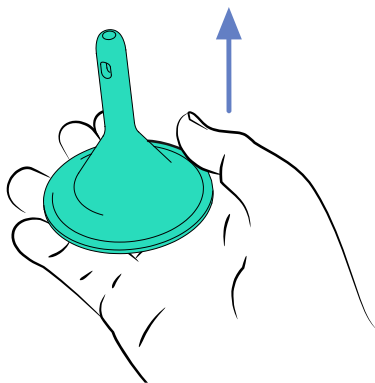
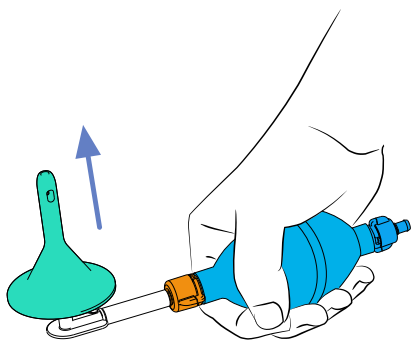
EN: Remove the cone from the elbow connector using the disposal bag supplied, the cone is designed to be used once. Dispose of according to local authority guidelines. Wash the pump & tubing/connector with mild soapy water and allowed to dry thoroughly, if you detach the elbow connection from the pump this will help the pump to dry out.

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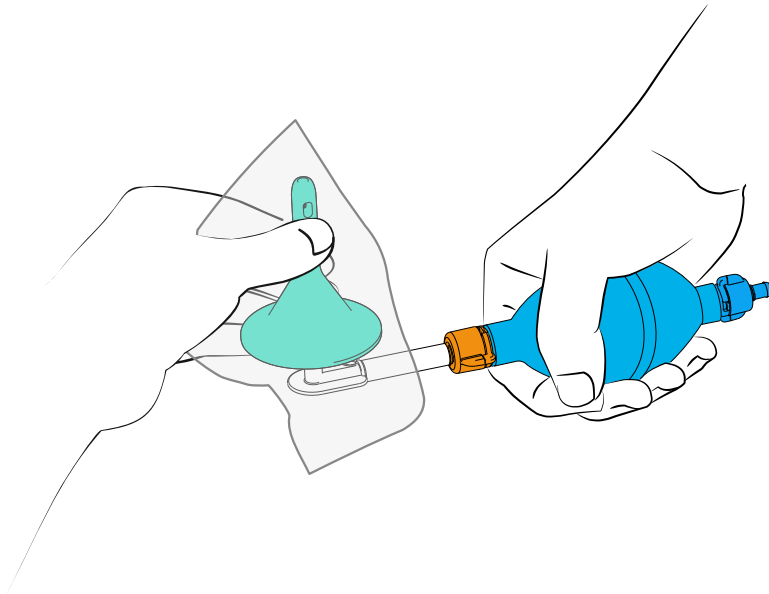EN: Aquaflush Anal Irrigation must only be used by the person that it has been prescribed for. The prescription should be written by a specialist healthcare professional after a thorough assessment has been made of the named patient.

CAUTIONS: The following list does not rule out anal irrigation as a bowel management programme, but you must discuss your general health with your specialist health care professional so that they can ensure that it is the right treatment for you. Please note that children under the age of 5 must be reviewed by their Paediatric Specialist before commencing assisted anal irrigation.

- >Inflammatory bowel disease (e.g. Crohn's disease or Ulcerative Colitis and Ischaemic Colitis)
- >Rectal pain or bleeding e.g. anal fissure, severe hemorrhoids
- >Diverticular Disease
- >Autonomic Dysreflexia
- >Recent spinal cord shock following spinal trauma
- >Radiation therapy in the abdominal or pelvic region
- >Abdominal or anal surgery
- >Recent colonic biopsy or endoscopic polypectomy
- >Cancer in the abdominal or pelvic region.
- >Long term steroid therapy, Anticoagulant Therapy
- >Coronary Disease
- >Pregnancy
- >Nursing mothers

WARNINGS: You must consult a physician if any of the following occur:

- >Blood in faeces, weight loss, abdominal pain.
- >Significant changes in the frequency, colour and consistency of the stools.

Note: Bowel perforation is an extremely rare, but a serious and potentially lethal complication to anal irrigation and will require immediate admission to hospital, often requiring surgery. Contact your doctor immediately, if during or after anal irrigation you experience any of the following symptoms:

- >Severe and sustained abdominal pain or back pain, especially if combined with fever
 - > Sustained anal bleeding

SAFETY: Ensure you store Aquaflush Systems away from the reach of small children to avoid risk of strangulation from tubing, suffocation from plastic bags or choking in the event of small components coming loose from the product.

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VitalCare Ltd.
VitalCare (Nanjing) Co., Ltd.