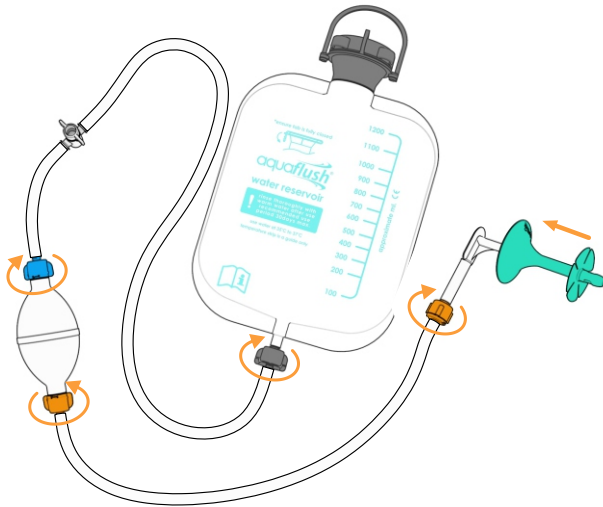
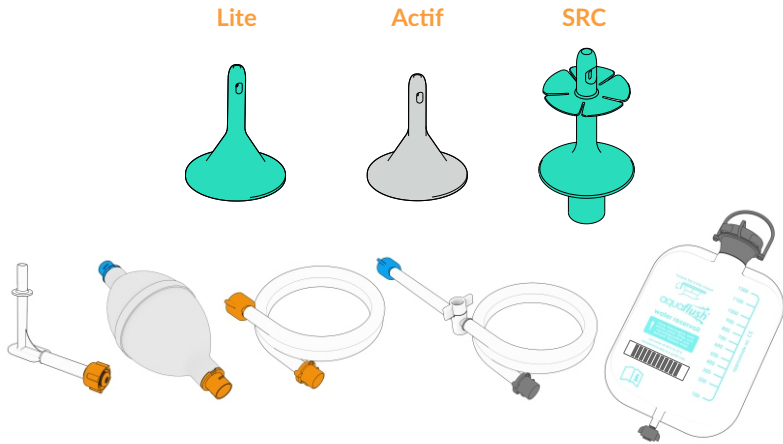
**SOLUTIONS  
WITH YOU  
IN MIND**

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Trans-anal Irrigation Systems: Instructions for Use  
Trans-anal Irrigasjonssystem: Brukerveiledning  
Transanal irrigations system: Brugsanvisning  
Transanal Irrigationssystem: Bruksanvisning  
Suolihuuhtelujärjestelmä: Käyttöohje

**EN:** Assemble the product as shown above, match the coloured connectors together and push the cone onto the elbow connector.

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**EN: Thank you for choosing an Aquaflush product.** Aquaflush Transanal Irrigation (TAI) systems are designed to be simple, easy to use and an effective way for you to control your bowel management routine with the minimum of fuss. Please read this document carefully, including the warnings and precautions, if you have any questions please contact us or discuss any concerns with your healthcare professional who will also provide you with some training before commencing this treatment.

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# 1

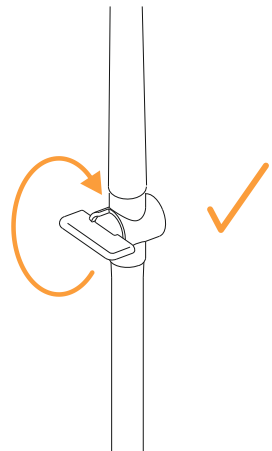
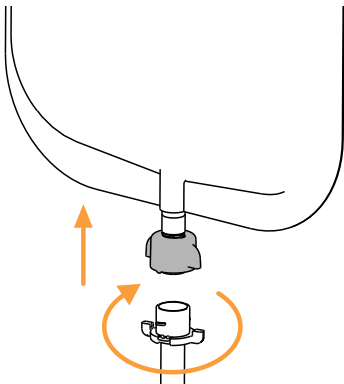
**EN:** Ensure you have everything ready to hand and wash your hands thoroughly. Before starting attach the pump tube to the water bag and close the in-line tap - ensure the tap head is positioned perpendicular to the tubing as shown in the diagram.

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## 2

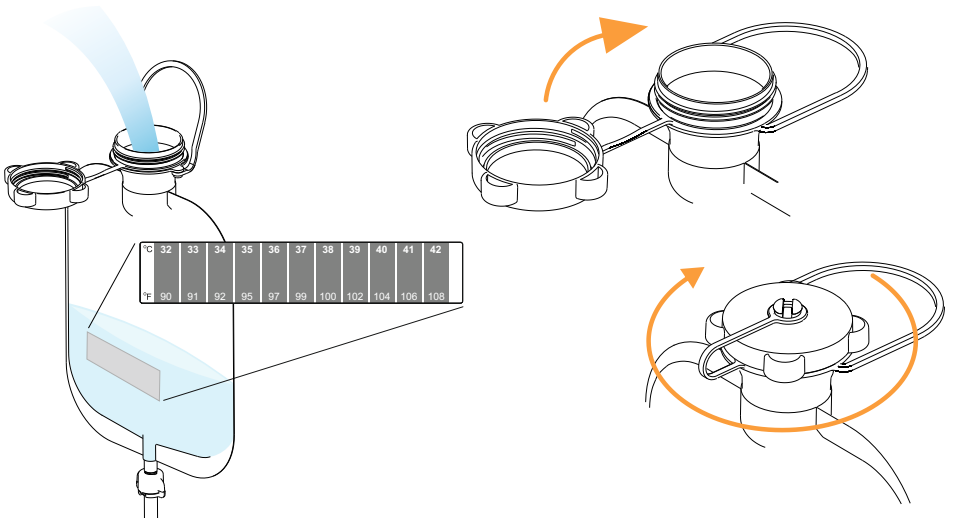
**EN:** Unscrew the waterbag cap & fill with water that is between 35° to 37° degrees centigrade (body temperature). If the water is too cold it may cause cramps, if it is too hot there is a danger of trauma to the bowel lining. Fold the bag at the water line to expel any air, before closing the cap securely. This will help ensure that no air is trapped in the bag and therefore avoid pumping air into the system.

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### 3

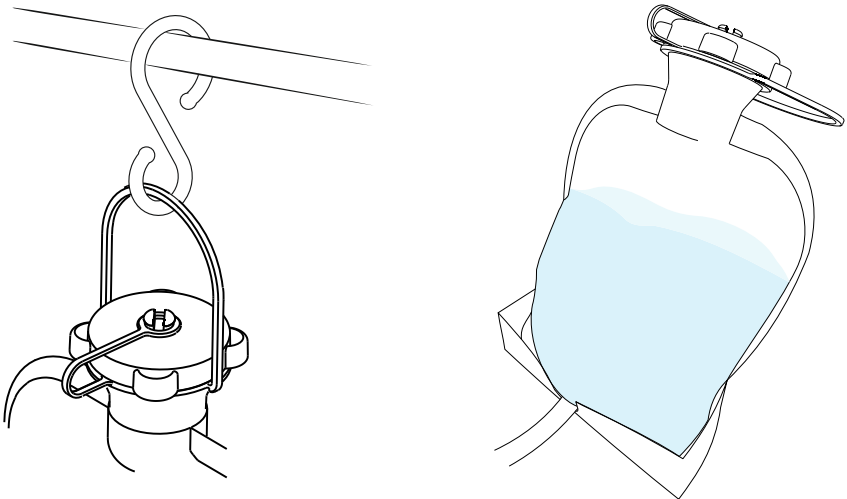
**EN:** The waterbag can be placed on a flat surface or clean floor using the bag stand provided. The waterbag also comes with a built in hanger & separate 'S' hook. If hanging, the bag should be located above the height of the toilet to avoid getting air into the system. If you use this method the system can free-flow so you do not need to pump. Please note that you can only free flow the water if the bag is above shoulder height to the person doing the irrigation.

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
**EN: Prime the system.** Open the tap while holding the cone over the sink or toilet allowing water to pass through the tubing and pump, squeeze the pump a few times to ensure there are no air bubbles trapped in the tubing/pump, it is important that you do not pump air into your bowel so always prime the system as described here. **Close the tap.**

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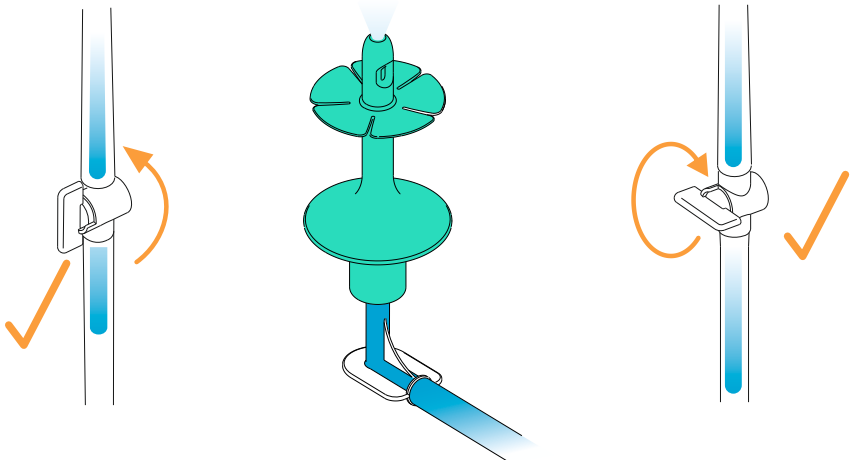
**EN:** Hold the cone with the tubing between your fingers and along the back of your hand. Apply lubricant to the tip and shaft of the cone, you can also apply some to the rectum using a gloved finger if required. Seated on the toilet, carefully push the cone into the rectum, if your are using the SRC cone pull back slightly so the petals sits firmly in the rectum. Some people may need to be taught the correct angle of insertion. Open the tap completely.

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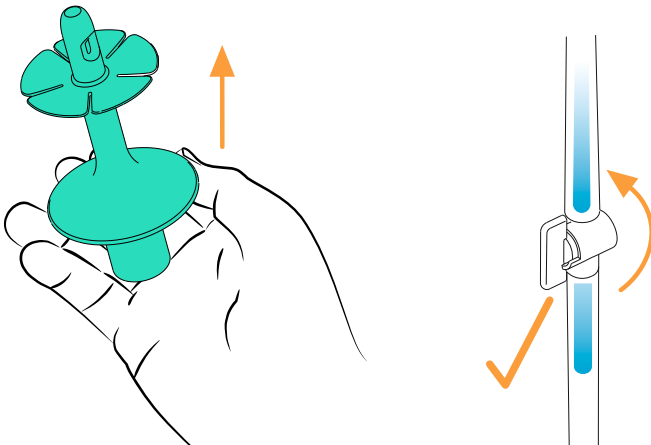
**EN:** Gently pump the volume of water advised by your healthcare professional into the rectum by monitoring the volume scale on the water bag [usually 300ml to 500ml]. If any discomfort is felt, stop pumping and close the tap immediately. When you have finished pumping, remove the cone from the rectum and remain seated on the toilet. The bowel may take up to 30 minutes to empty. Wait about 5 minutes after everything appears to have been expelled.

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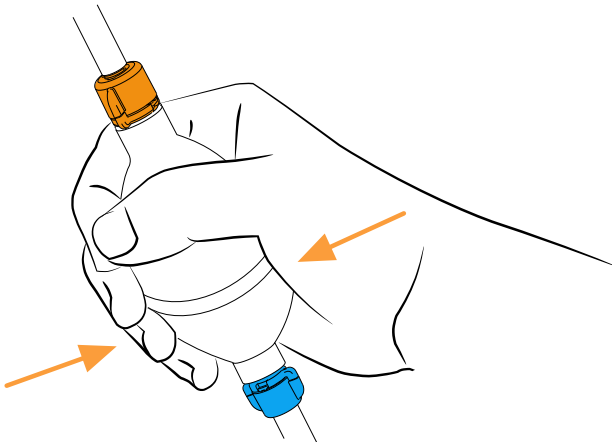
**EN:** Close the tap and remove the cone from the elbow connector using the disposal bag supplied, the cone is designed to be used once. Dispose of according to local authority guidelines. Wash the water bag, tubing and pump with mild soapy water and hang to dry with the cap and tap left open and the pump disconnected from the water bag.

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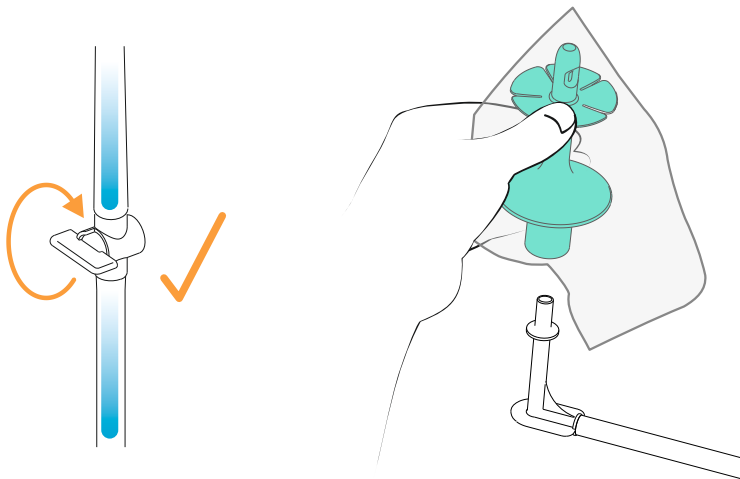**EN:** Aquaflush Anal Irrigation must only be used by the person that it has been prescribed for. The prescription should be written by a specialist healthcare professional after a thorough assessment has been made of the named patient.

**CAUTIONS:** The following list does not rule out anal irrigation as a bowel management programme, but you must discuss your general health with your specialist health care professional so that they can ensure that it is the right treatment for you. Please note that children under the age of 5 must be reviewed by their Paediatric Specialist before commencing assisted anal irrigation.

- >Inflammatory bowel disease (e.g. Crohn's disease or Ulcerative Colitis and Ischaemic Colitis)
- >Rectal pain or bleeding e.g. anal fissure, severe hemorrhoids
- >Diverticular Disease
- >Autonomic Dysreflexia
- >Recent spinal cord shock following spinal trauma
- >Radiation therapy in the abdominal or pelvic region
- >Abdominal or anal surgery
- >Recent colonic biopsy or endoscopic polypectomy
- >Cancer in the abdominal or pelvic region.
- >Long term steroid therapy, Anticoagulant Therapy
- >Coronary Disease
- >Pregnancy
- >Nursing mothers

**WARNINGS: You must consult a physician if any of the following occur:**

- >Blood in faeces, weight loss, abdominal pain.
- >Significant changes in the frequency, colour and consistency of the stools.

Note: Bowel perforation is an extremely rare, but a serious and potentially lethal complication to anal irrigation and will require immediate admission to hospital, often requiring surgery. Contact your doctor immediately, if during or after anal irrigation you experience any of the following symptoms:

- >Severe and sustained abdominal pain or back pain, especially if combined with fever
  - > Sustained anal bleeding

**SAFETY:** Ensure you store Aquaflush Systems away from the reach of small children to avoid risk of strangulation from tubing, suffocation from plastic bags or choking in the event of small components coming loose from the product.

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VitalCare Ltd.
VitalCare (Nanjing) Co., Ltd.