

FRONT



- Always consult your Healthcare Professional before using this product.
- The product life is 30 days maximum intermittent use and should be replaced after this.
- This product should only be used by one patient.


PLEASE WASH YOUR HANDS THOROUGHLY BEFORE USING THIS PRODUCT

1. Using a clean, sharp pair of scissors, cut the stopper to the length suggested by your Healthcare Professional. Cut at the ribbed section to ensure a smooth edge for insertion.



2. Gently insert the stopper into the tract, a small amount of water-based lubricant can be used to help. The cap of the stopper should sit flush on your skin.

3. Ensure your skin is dry and then apply the dressing over the cap of the stopper to hold it in place.

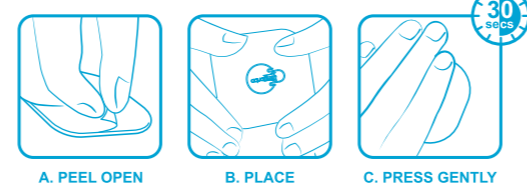

4. To remove the stopper, gently peel off the dressing and then use the small tab to slowly pull-out the stopper.

5. Before re-using the stopper, you must clean it thoroughly and dry it as instructed by your Healthcare Professional.

RM-SSIFUV2

BACK

Store in a cool dry place and use stopper for no more than 30 days

Size	Order Code	PIP Code
8fr	ACEM8	401-8941
10fr	ACEM10	401-8958
12fr	ACEM12	401-8966
14fr	ACEM14	401-8974
5x5cm	AFD30	401-8925
8fr	ACE8/100	401-8891
10fr	ACE10/100	401-8909
12fr	ACE12/100	401-8917
14fr	ACE14/100	401-8933

Stoma Stopper Monthly Pack  
 Stoma Stopper Monthly Pack  
 Stoma Stopper Monthly Pack  
 Stoma Stopper Monthly Pack  
 Ultra-thin Hydrocolloid Dressings x 30  
 Stoma Stopper 100mm [adjustable] + Dressing  
 Stoma Stopper 100mm [adjustable] + Dressing  
 Stoma Stopper 100mm [adjustable] + Dressing  
 Stoma Stopper 100mm [adjustable] + Dressing

Renew Medical UK  
 www.renew-medical.uk  
 support@renew-medical.uk  
 0800 542 0814

SINGLE PATIENT ONLY

Stoma Stopper IFU 14.07.20	
RM-SSIFUV2	APPROVED: